TRAYDEX 45

LED Light Curing Machine

Instructions for use

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INNOVATION

**SAVE TIME, SAVE COST,
MULTIPLE FUNCTIONS**

Dear Customer,

Thank you for choosing DENTMATE TRAYDEX 45.

A lot of researches & developments have gone into the manufacturing of this product. We sincerely hope that it will give you many years of trouble-free use. Please read and understand all the instructions before using this equipment.

Save this instruction for use for your reference.

The instruction for use is subject to change without further notice.

DENTMATE TECHNOLOGY CO., LTD.

8F, No. 8-11, Sec. 1, Zhongxing Road, Wugu District,
New Taipei City 24872, Taiwan

TEL : +886 2 8976 9226 FAX : +886 2 8976 9236

WEBSITE : www.dentmate.com.tw

EMAIL : info@dentmate.com.tw

Table of Contents

1. Symbols Used	2
1.1 In these instructions for use	2
1.2 On the product / packaging	2
2. Safety Information	3
2.1 Installation and connection	3
2.2 Operation	4
2.3 Maintenance	5
3. Product Information	6
3.1 Indications for use	6
4. Package And Part Name	7
4.1 Main unit part name	7
4.2 Table of contents	7
5. Functions	8
5.1 Instruction for touch panel	8
5.2 IDT (Intensity Duty-cycle Timer) setting	8
5.3 Basic operation	9
6. Troubleshooting	11
6.1 Instructions for fuse replacement	11
7. Maintenance And Cleaning	12
7.1 Cleaning inner chamber	12
7.2 Cleaning outer case	12
8. Product Specification	13
9. Warranty	14
10. Declaration Of Conformity	14

1. Symbols Used

1.1 In these instructions for use

Warning




Indicates a potentially hazardous situation that could result in serious injury or death.

Caution

Indicates a potentially hazardous situation that may result in minor / moderate injury or damage of property.

1.2 On the product / packaging

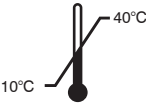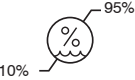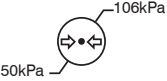 Serial number	 Catalogue number
 Manufacturer	 Date of manufacture
 Temperature limits	 Consult the Instructions for use
 Storage humidity range	 Atmospheric pressure for storage
 CE marking	 Store in a dry place
 Danger Electricity	 Hot Surface

2. Safety Information

2.1 Installation and connection

Warning

- Please install the product at least 5 cm away from the wall. In addition, do not place any objects above the product. Please keep the product away from any heating appliances. Overheating the product may cause fire and/ or electrocution.
- Do not block the ventilators on the left, right, and back side of the product. Blockage may cause fire and/ or breakdown because of rises in the inner temperature of the product.
- Keep the product at a location away from water.
- Keep the product at a location without excess humidity and temperature.
- Do not keep the product at a location for chemicals or a location where gas is released.
- Do not keep the product at a sloping location. Do not shake or impact the product.
- Use only AC100-240 V power-supply voltage. Using the wrong voltage input may cause fire and/or electrocution.
- Do not place heavy objects on the power cord. Damage to the cord may cause fire and/ or electrocution.
- Do not disassemble this product. Unauthorized disassembly of this product may cause electrocution and/ or breakdown.
- Do not damage, bend, twist or pull the power cord. Doing so may cause fire and/ or electrocution.
- Ensure earthing of terminal outlet and power socket. Electricity leak in the main unit might cause electric shock and/ or fire.
- Pay attention to the frequency of the power supply 、 power voltage and allowable current value (or consumed power.)

2.2 Operation

Warning

- This product should be operated by qualified people only.
- If the product is in an abnormal state, such as emitting smoke and/or making irregular noise, please switch off the power switch immediately. Pull out the plug and contact the distributor.
- If water leak inside the product, switch off the power switch immediately. Pull out the plug and contact the distributor. Continue to operate this product may cause fire and/ or electrocution.
- Never touch the power plug with wet hands to avoid electric shock.
- Never handle any liquid or metal power in the curing chamber to avoid electric shock or fire caused by internal circuit failure.
- Do not disassemble this product. Unauthorized disassembly of this product may cause electrocution and/ or breakdown.
- Please wear goggle during operation; protect eyes and skin away from UV exposure.
- To stop operation, please press the STOP button and confirm the light is off before opening the door and taking out the object.
- To stop operation by opening the door, please open the door slightly and confirm the light is off before opening the door entirely and taking out the object.

Caution

- Do not look inside the product during operation.
- Do not push the Start-Stop button while the door is opened.
- Do not operate the product when the door is opened, as it may cause damage to your eye.

2.3 Maintenance

Warning

- If this product is dropped, broken or damaged, switch off the power switch. Then pull out the plug and contact the distributor. Continue to operate the product may cause fire and/ or electrocution.
- Be sure to pull out the plug when maintaining the product.
- When moving the product, please switch off the power switch and pull out the plug.
- When pulling out the plug, do not pull out the power cord.
- When checking, wiping or replacing fuses, always turn off and unplug the unit. Leave the unit off for at least 10 minutes to ensure that the unit has cool off completely.

Caution

- Remember to have regular inspections of the equipment and parts.
- If the product will not be used for a long time, please pull out the plug for safety.

3. Product Information


- Save time
Fast curing High power LED, broadband LED, save fabricating time.
- Save money
Stable and high power LED with constant current circuit, avoid the cost and hassle of replacing UV fluorescent light tube.
- Multiple functions
Users could set up the curing details easily to gain process efficiencies by Intensity Duty-Cycle Timer (IDT) settings.
- Post-curing for 3D printing
High power Ultra Violet A (UVA) LEDs is including, which results in a quick and uniform curing cycle.

3.1 Indications for use


This product is a light curing machine for laboratory and professional use only.

4. Package And Part Name

4.1 Part name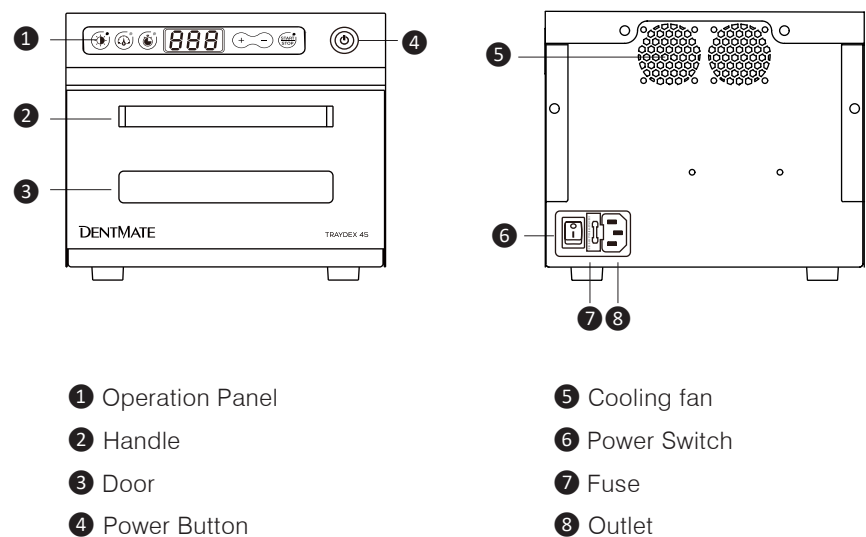

4.2 Components

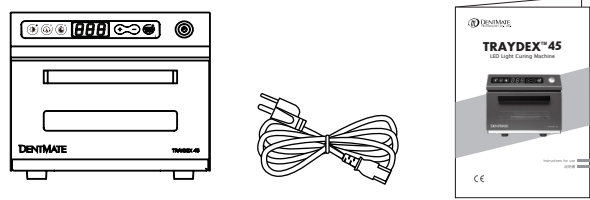Description	Quantity
TRAYDEX 45	1
Power cord	1
Instructions for use	1

5. Functions

5.1 Instruction for touch panel

- ① Adjust the light intensity from Level 1 to Level 16.
- ① Adjust the duty cycle (See chapter 5.2) from Level 1 to Level 10 (consistent brightness)
- ① Adjust the working time between 5 seconds and 995 seconds.
- ② Display: Indicate numerical value and remaining seconds.
- ③ + : Increase the level of intensity, duty cycle and working time. Hold and press the button to fast-forward.
- ④ - : Decrease the level of intensity, duty cycle and working time. Hold and press the button to rewind.
- ⑤ : Start and stop curing

5.2 IDT (Intensity Duty-Cycle Timer) settings


- Intensity Setting

Depending on the strength, thickness, and shape of the material, operator can set the light intensity from Level 1(lowest) to Level 16(strongest). Intensity range: 7018 –112295 mW/cm² when LED current at 43.8-700mA.

Suggestion: When curing flat and/ or thin material, lower the intensity level with longer timer to optimize the overall strength of the material.

- Duty Cycle

Duty cycle is the ratio between pulse duration and the period(Picture below shows the difference between differnt cycle). Depending on the shrinking rate of the material, operator can set the duty cycle from level1(10%) to level 10(100%).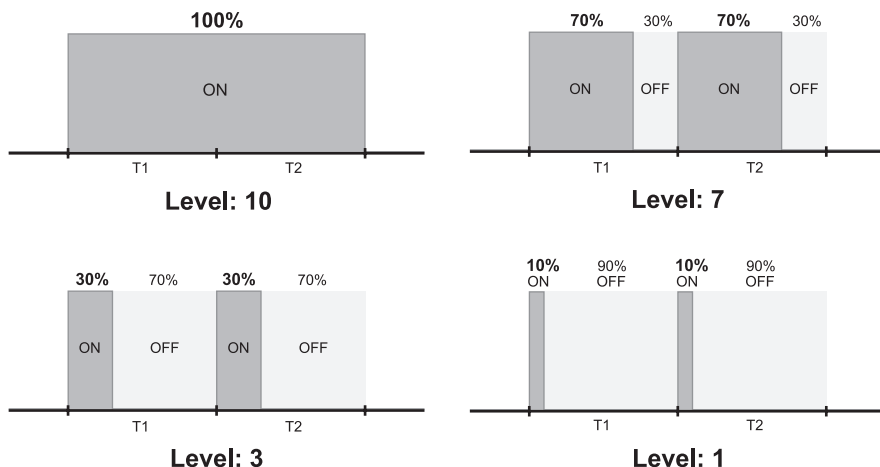


Suggestion: When curing small and/ or thin material, lower the duty cycle level to reduce the shrinkage deformation rate.

- Curing Time

Set the time between 5 - 995 seconds with 5 seconds interval.

5.3 Basic Operation

1. Turn on the Power Switch.

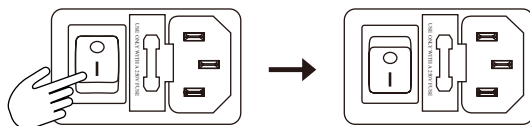
2. Press  to power on.
3. Press , ,  to set up IDT.
4. Press **START** to cure.

5.3.1 Pause curing during the operation


Press **START/STOP** to pause curing.

The **START/STOP** button will start flashing when paused.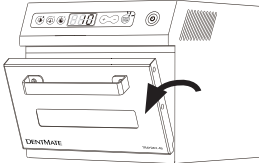

Note, curing will be paused once the door is opened. The **START/STOP** button will start flashing as well.


5.3.2 Increase/decrease curing time when pause curing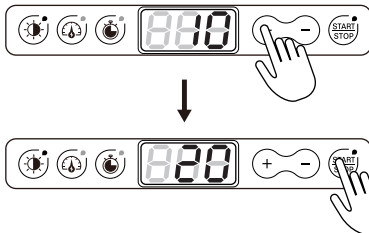

Press + to increase or press – to decrease curing time.

After adjusting the curing time, press **START/STOP** to restart.

5.3.3 Reset IDT when pause curing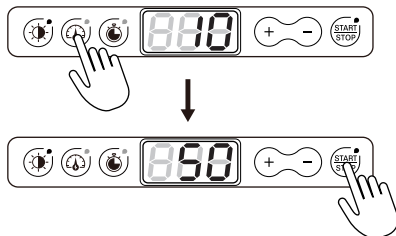

Press the buttons to reset IDT setting.

After resetting, press **START/STOP** to restart it




5.3.4 Memory mode

1. Make sure the power is on
2. Press    to set up the terms of curing.
3. Hold and press the button . After hearing 2 beeps, the system will memorize the IDT setting and save it as the new default.


6. Troubleshooting

Trouble	Possible Cause	Measures
Time display does not light up even if the power switch is turned on.	Blowout of fuse.	Replace with new fuse. Contact our customer service if the problem persist.
LED do not light up even if the START/STOP is pressed.	Door is not closed correctly.	Ensure the door is closed properly.
Er1	Overheat	Immediately turn off the unit. Contact our customer service and mention the error number on the display.


6.1 Instructions for fuse replacement

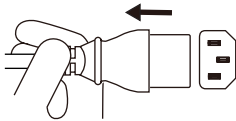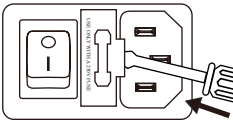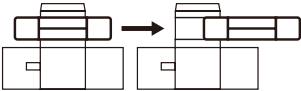
❶ Before replacing the fuse, turn off the power switch and pull out the plug.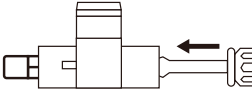


❷ Use flat head screw driver to unclench the cover shown above.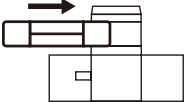


❸ Take the fuse out.

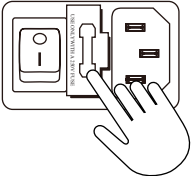
❹ Take out spare fuse at the bottom.

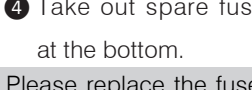❺ Change the fuse.

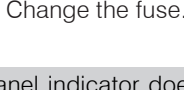❻ Put the fuse cover back.

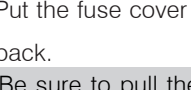* Please replace the fuse if the panel indicator does not work. Be sure to pull the power cord before replacement. After replacement, if there's still a problem with the panel indicator, please contact customer and service for repair.

7. Maintenance And Cleaning

- Remember to have regular inspections of the equipment and parts.
- In case of malfunction, do not attempt to repair the product by yourself. Contact customer and service for repair.
- Before maintenance, switch off the power switch and pull out the plug.





7.1 Cleaning inner chamber

- 1- When the inner chamber gets dirty, clean it with soft cloth moisture with alcohol or cleaner.
- 2- If resin adheres to any part of the curing chamber, please remove it prior to next curing.

7.2 Cleaning outer case

- 1- Do not use organic solvent including benzene and thinner to clean.
- 2- Gently wipe the unit with a soft cloth moisture with alcohol or cleaner.

8. Product Specification

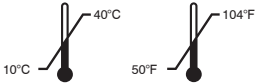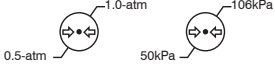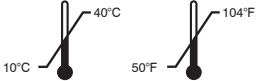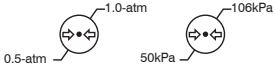Type of Information	Specifications
Device name	LED Light Curing Machine
Model name	TRAYDEX 45
Power input	AC 100-240 V / 50-60 Hz
Power consumption	70 W
Fuse	2 A
Quantity of LEDs	25
Wavelength of LEDs	375~470 nm
Curing time	5-995 seconds
Dimensions	L200 x W220 x H160 mm (max)
Weight	3 Kg
Operating environment	Indoor use only. Ambient temperature:  Relative humidity:  Atmospheric pressure: 
Storage and transport environment	Ambient temperature:  Relative humidity:  Atmospheric pressure: 

9. Warranty

DENTMATE TECHNOLOGY Co., Ltd. warrants the product to be free of manufacturing defects for a period of one year from the date of purchase; this is deemed as the date of the invoice. It could be repaired or replaced at its own discretion all equipment failures due to manufacturing defects. However,the followings are expressly excluded from the warranty:

- 1. Damage and/ or failure of the equipment caused by falling and/ or jolting during transportation after purchase and/or during the normal use.
- 2. Damage and/ or failure of the equipment caused by natural disasters, such as earthquakes, floods, lightning, pollution, incorrect electrical voltage and voltage spikes.
- 3. Any attempts to change or re-model the product will invalidate the warranty.

10. Declaration Of Conformity

 Manufacture	DENTMATE TECHNOLOGY CO., LTD. 8F, No.8-11, Sec 1, Zhongxing Road, Wugu District, New Taipei City 24872,Taiwan
Safety	IEC / EN 61010-1 3rd : 2010
EMC	EN 61326-1:2013

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